911 CANYON

880.01

CE

Regolamento (UE)
2017/745

IT

WWW.KONG.IT

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TABLE OF CONTENTS

1 - SYMBOLS AND SUPPORT	4
1.1 Symbols	4
1.2 Technical assistance	4
2 - GENERAL INFORMATION	5
3 - TECHNICAL FEATURES	6
3.1 Terminology and materials of the parts	6
3.2 Dimensions	7
3.3 Range	7
3.4 Optional components and spare parts	10
4 - SPECIFIC INFORMATION	11
4.1 Intended use	11
4.2 Assembly	11
4.3 Positioning and immobilising the patient	14
4.4 Transporting the patient	18
4.5 Transporting of the stretcher	20
5 - MAINTENANCE AND REPAIR	21
5.1 General Information	21
5.2 Maintenance	21
5.3 Repair	21
6 - STORAGE	21
7 - CHECKS, INSPECTIONS AND REVISIONS	23
7.1 Checks	23
7.2 Inspections	23
7.3 Revisions	23
8 - DURATION AND WARRANTY	24
8.1 Duration	24
8.2 Disposal	24
8.3 Warranty	24
8.4 Law obligations	24
9 - LABELLING AND SYMBOLS	25
9.1 Medical Device Label	25
9.2 Symbols	25
10 - DOCUMENTS	26
10.1 Inspections and servicing register	26
10.2 Maintenance and repair records	27
10.3 EU Declaration of conformity	28
11 - LIST OF APPLIED AND REFERENCE STANDARDS	29
11.1 Applied standards	29
11.2 Standards used as reference	30

CHAPTER

1

SYMBOLS AND SUPPORT

1.1 SYMBOLS

For the sake of comfort and clarity while reading this manual, below are the symbols used for handling important warnings for a proper and safe use of the device.

REQUIREMENT FOR PROPER USE

It identifies the presence of information for proper use of the device.

INFORMATION REQUIREMENT

It identifies the presence of useful and general information which guides the reader towards a conscious use of the device and/or performance of actions.

It identifies that the Medical Device is manufactured, designed and produced in compliance with the provisions of the General Safety and Performance Requirements of Regulation (EU) 2017/745 (Class I medical device, in compliance with classification rule 1 as indicated in the attachment VIII).

1.2 SUPPORT

For information please contact Kong Customer Support Service by:

- telephone +39 0341 630506
- fax +39 0341 641550
- email: safetycare@kong.it

or write to KONG S.p.A. – Via XXV Aprile, 4 – 23804 Monte Marenzo LC - ITALY.

To facilitate support operations, please always communicate or state the serial number (SN) indicated on the label fixed to the Medical Device.

CHAPTER 2

2 GENERAL INFORMATION

The information provided by the manufacturer (hereinafter referred to as information) must be read and clearly understood by the user prior to using the Medical Device (MD hereafter). The information regards the description of the features, performance, assembly, disassembly, maintenance, preservation, disinfection, etc. of the device. Even though the information offers tips on use, this information shall not be considered as a user manual under actual conditions of use.

WARNINGS AND LIMITATIONS OF USE:

- this device shall be strictly used by people who are physically fit, trained (instructed and taught) to use the device and with specific experience regarding moving the patient or, during training activities, by people under direct supervision of the trainers/ supervisors who guarantee the safety thereof,
- do not use the device before fully reading and understanding this user manual,
- strictly follow the manufacturer's information, improper use of the device is hazardous,
- modifying and/or repairing the device is strictly forbidden,
- all checks described in chapter 7 must be carried out prior to and after using the device. In case of any doubt on the efficiency of the device, the user must replace it immediately,
- non-compliant use, deformations, falls, wear, chemical contamination, exposing textile/plastic components/devices to temperature below -30°C or above

- +50°C and metal components/devices to temperatures exceeding 100°C are some examples of causes that can reduce, limit and end the life of the device,
- Prior to any rescue operation, be keen not to exceed the capacity indicated in paragraph 3.3,
- in order to reduce risks of exposure to / transmission of infectious diseases, clean and disinfect the device as indicated in chapter 5,
- improper use of the patient immobilisation systems can jeopardise the safety of the patient,
- always check the compatibility of the devices used alongside the device by consulting the relative manufacturer's information,
- use of spare parts or optional components different from the ones indicated in paragraph 3.4 can be hazardous,
- do not expose the device to sources of heat and at contact with chemical substances. Reduce exposure to direct sunlight as much as possible. At low temperatures and in the presence of humidity, formation of ice could reduce flexibility and increase cutting and abrasion-related risks on textile and synthetic devices,
- Report to the manufacturer and the competent authority of the member state where the user and / or patient is established any serious incident occurring in relation to the MD.

All our devices are tested/ checked piece by piece in compliance with the procedures laid down by the Quality System certified in accordance with the UNI EN ISO 9001 standard. Laboratory tests, testing, information and standards do not always reproduce the practical result. Thus, the results obtained under the actual conditions of use of the device in the natural environment may differ, even considerably at times. The best information lies in the continuous practical use under the supervision of skilled/ expert/qualified people.

Warning: not suitable for use in an ATEX environment (Directive 94/9/CE)

CHAPTER 3

TECHNICAL FEATURES

3.1 TERMINOLOGY AND MATERIALS OF THE PARTS

- A – Aluminium alloy frame, can be disassembled into two parts,
- B – Protective shells made of GRP,
- C – Aluminium alloy handles,
- D – Polyurethane padding,
- E – Waterproof canvas in trilaminate,
- F – Aeration valve in ABS,
- G – Cranial protection in Lexan®,
- H – HRP head immobiliser,
- I – Polyester shoulder straps,
 - Mattresses in polyurethane foam,
 - Polyester webbing,
- L – Hanging kit,
- M – Padded straps.

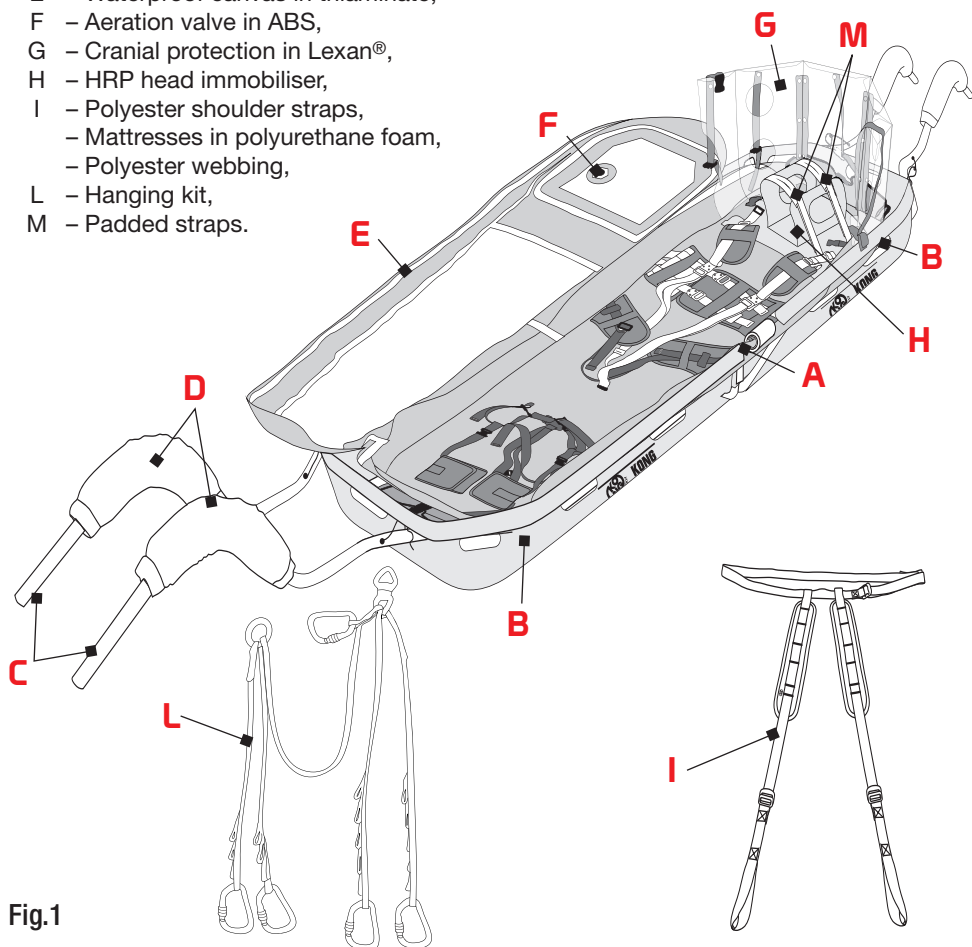
Fig.1

3.2 DIMENSIONS

Length without handles:	220 cm
Length with handles:	350 cm
Width:	60 cm
Overall weight (fig.1):	23 kg

3.3 CARRYING CAPACITY

The table below defines the capacities of the “911 CANYON” stretcher, based on the method of use deemed most appropriate by the rescue operator, the outcome of the laboratory tests and recommended safety coefficients.

How to use	Capacity
Carriage with handles/sun loops on the side of the frame (fig. 2-3)	150 kg
Lifting and lowering using rescue helicopters (fig. 4) <i>with hanging kit</i>	
Transport by cable car and airlines (figg. 4-5) <i>with hanging kit</i>	
Vertical descents and recoveries (fig. 6) <i>by rope</i>	

Prior to any rescue operation, ensure not to exceed the indicated capacity.

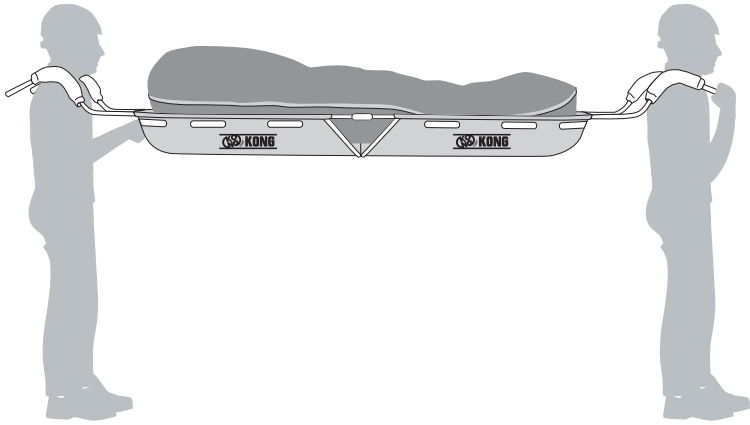
Fig. 2

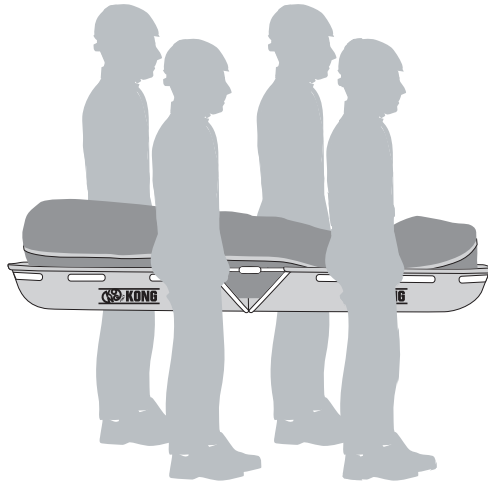
Fig. 3

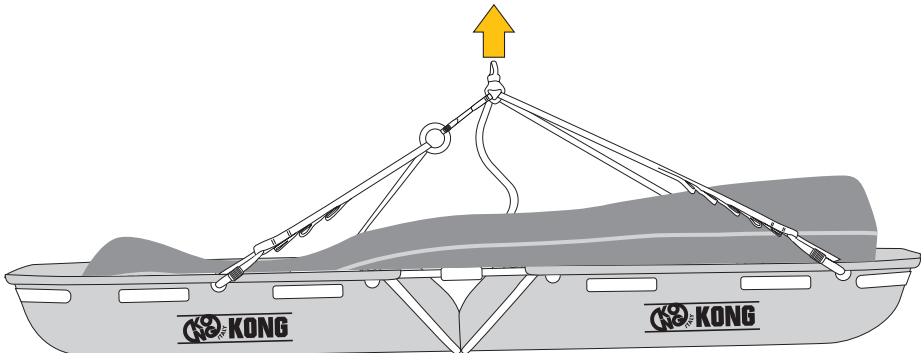
Fig. 4

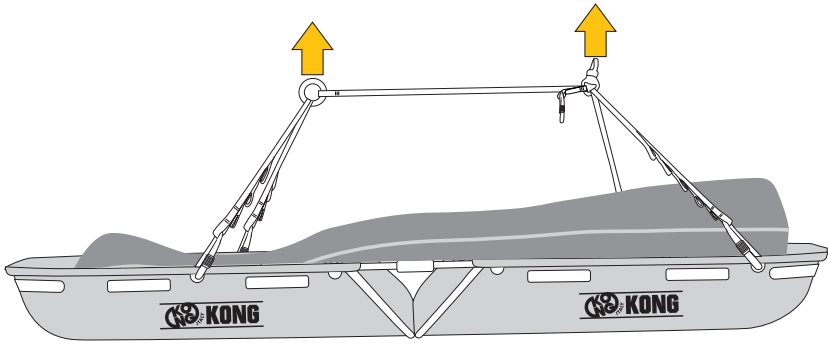
Fig. 5

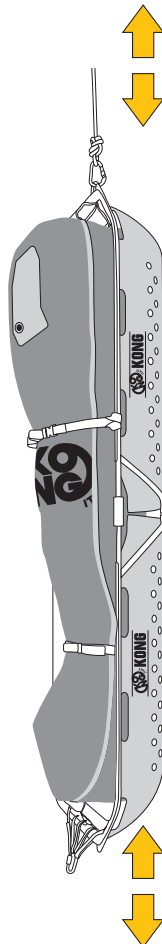
Fig. 6

3.4 OPTIONAL COMPONENTS AND SPARE PARTS

3.4.1 Optional components

880010300KK 911 BAG

Winch carriage bag

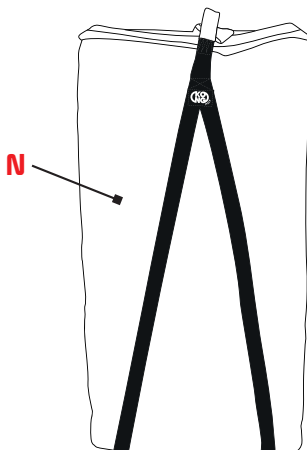
Fig.7

3.4.2 Spare parts

- D - Paddings for handles,
- G - Cranial protection,
- I - Shoulder straps for stretcher,
- L - Suspended,
- M - Rope bag.

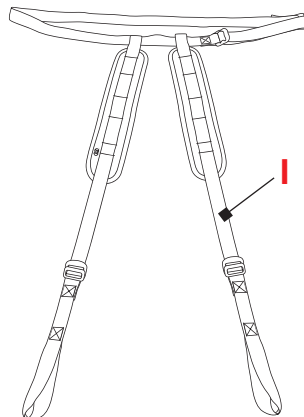
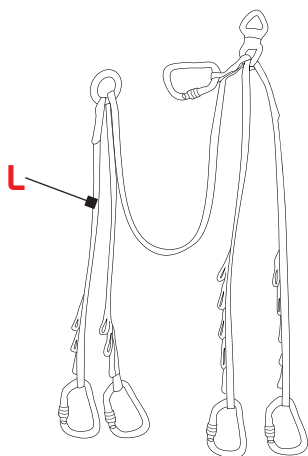
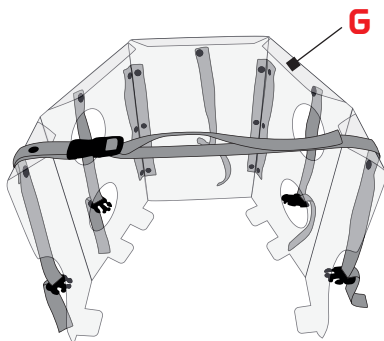
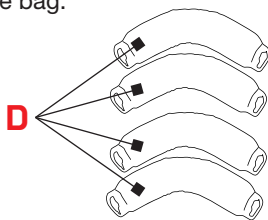
Fig. 8

CHAPTER 4

SPECIFIC INFORMATION

4.1 INTENDED USE

The “**911 CANYON**” stretcher is a MD particularly suitable for recovering and transporting a patient in a gorge, even if immobilized on the “**VACUUM**” vacuum mattresses. Decisions concerning moving and immobilising the patient, as well as the duration, method to be used and combination with other devices, shall be taken and executed by expert and trained personnel only.

4.2 ASSEMBLY

Assemble the two parts of the structure by means of the following operations:

- a) rotate the two safety rods (Fig. 9),
- b) lift and turn the two locking rods (Fig. 10),

Fig. 9

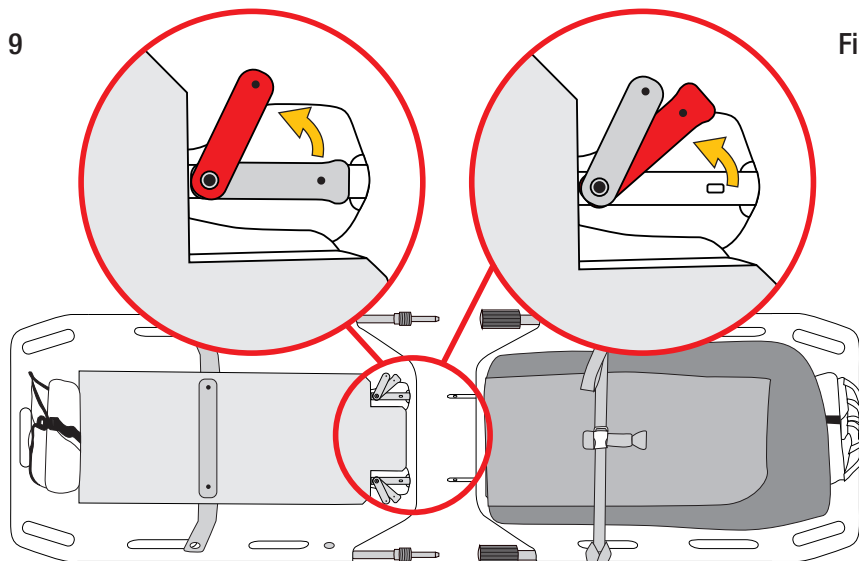
Fig. 10

c) connect the two parts of the structure by means of the frame pins,

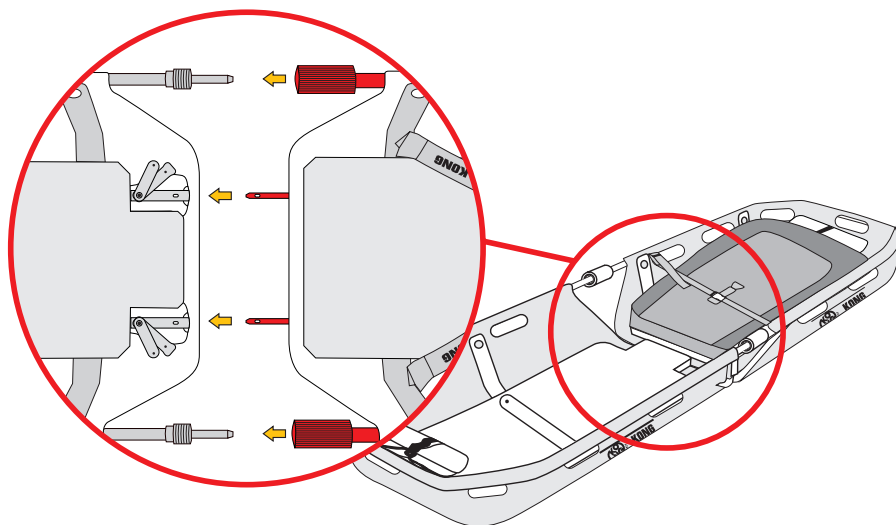
Fig. 11

- d) reposition the two locking rods in their respective seats (Fig. 12),
- e) reposition the two safety rods (fig. 13) and check the tightness of the coupling,
- f) Screw the two rings together completely (fig. 14).

Fig. 12

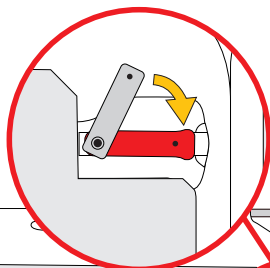
Fig. 13


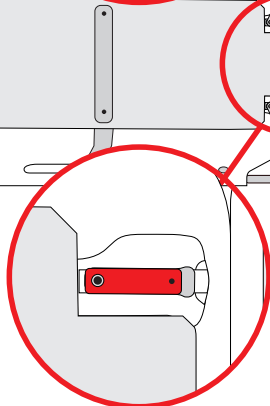
Fig. 14

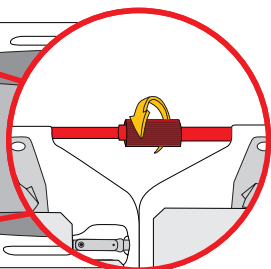g) attach the watertight sheet to the lower part of the structure (feet) by means of the automatic buckle.

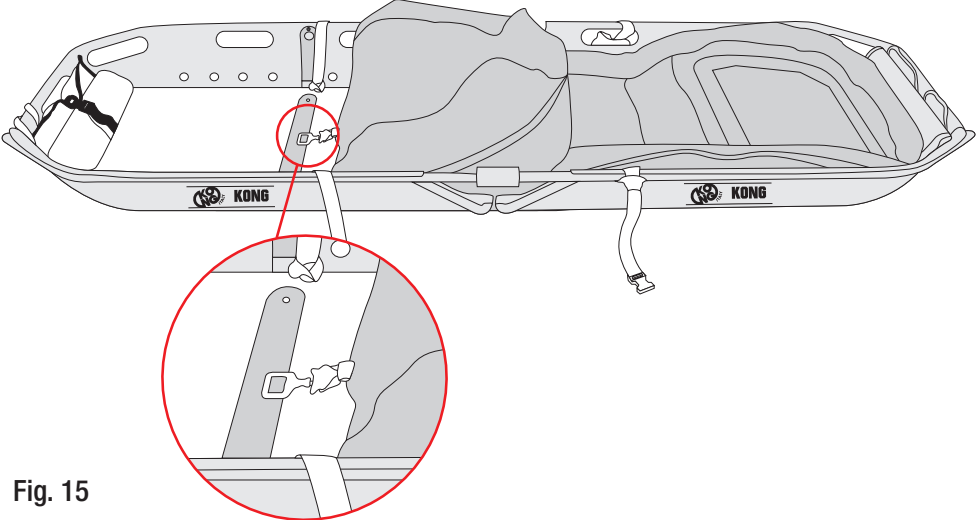
Fig. 15

h) lay down the watertight sheet.

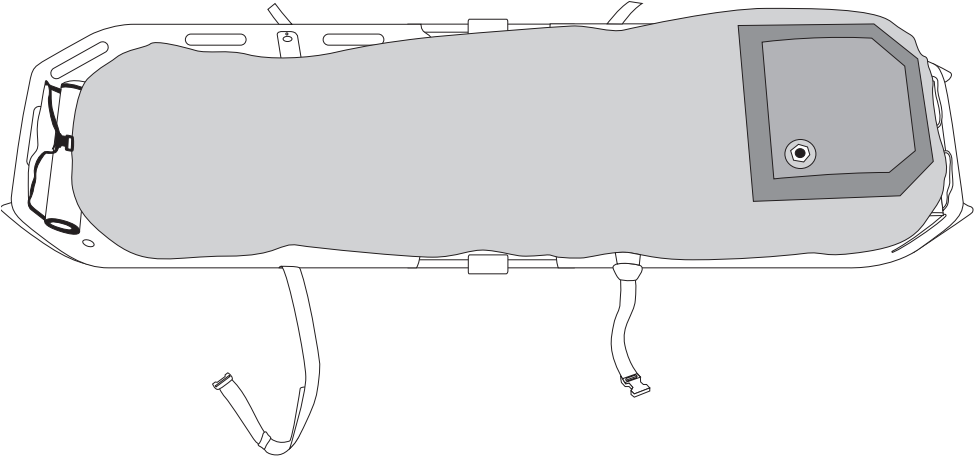
Fig. 16

4.3 POSITIONING AND IMMOBILISING THE PATIENT

With the scoop stretcher positioned on a flat and stable surface:

- a) release the buckles and open the outer straps,
- b) open the sheet completely, holding both edges of the zipper and avoiding tearing,

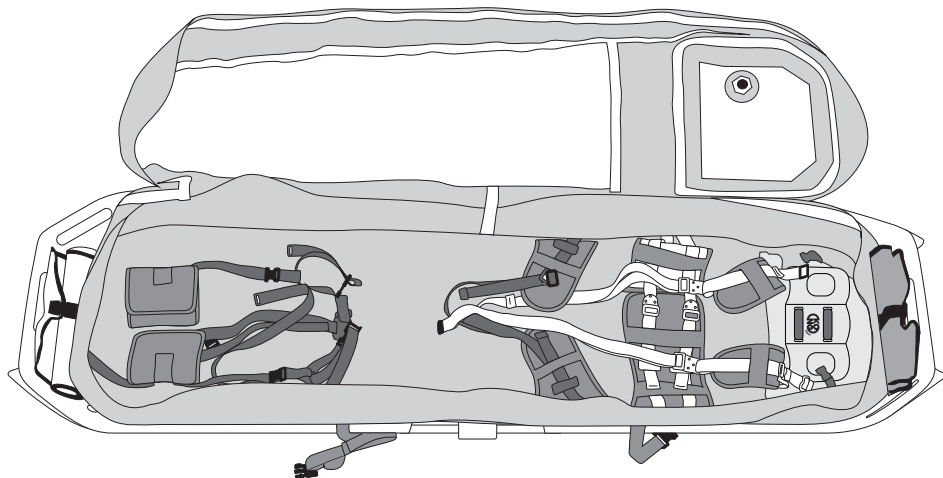
Fig. 17

- c) connect the cranial protection to the structure:
 - insert the fins into the slots provided (fig. 18),
 - close the nylon buckles and tension the webbing (fig. 19).

All fins must rest on the Lexan® worktop

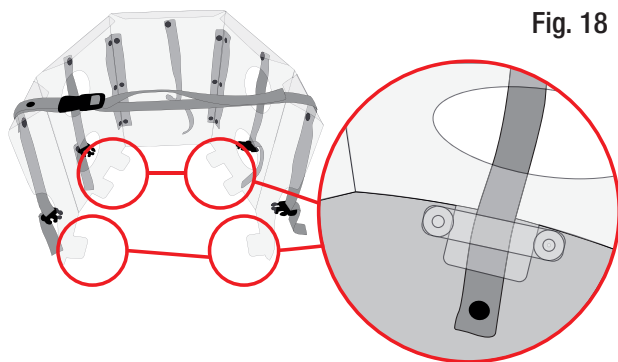
Fig. 18

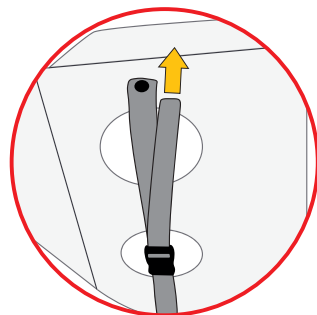
Fig. 19

d) release the buckles, open the locking harness, position and adjust it,

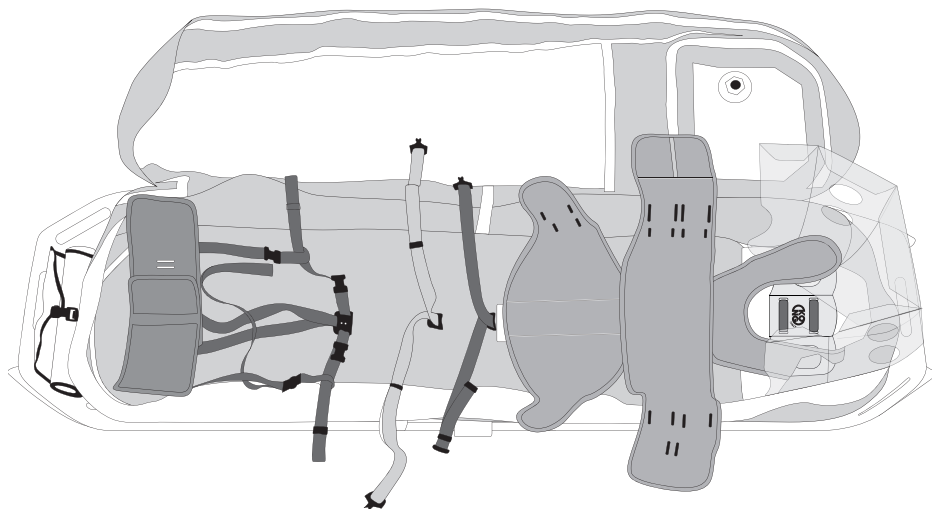
Fig. 20

e) place the patient in the stretcher,

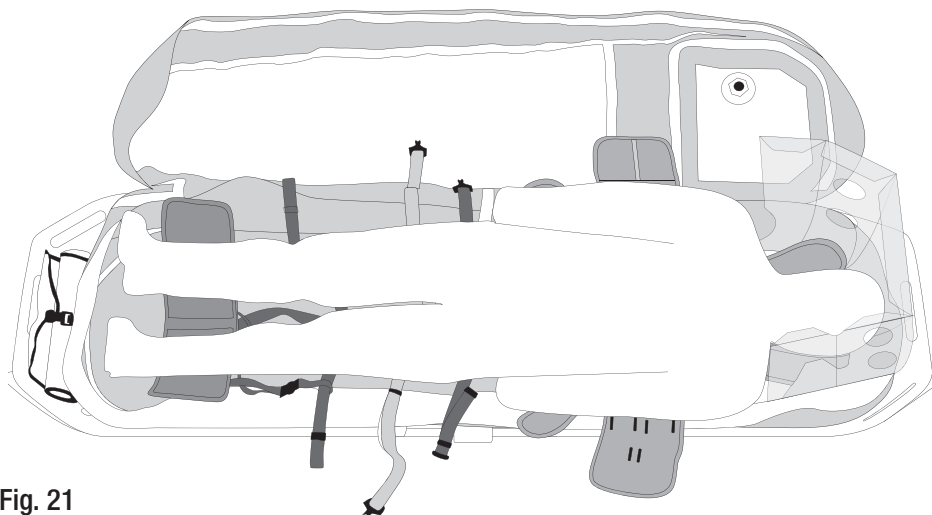
Fig. 21

f) secure the patient with the harness,

- This procedure should also be applied in the case of patients immobilised on the "VACUUM" vacuum mattress,
- the harness can also be used in part,
- the patient's arms are held by the tensioning of the outer straps.

Fig. 22

g) place the patient's head in the HRP head immobiliser and immobilise it using the padded straps (fig. 23),

The head immobiliser can also be used on patients immobilized with cervical collars.

During vertical descent and recoveries (Fig. 6), the use of the lower padded straps (M2), on patients without a cervical collar, can lead to the risk of strangulation.

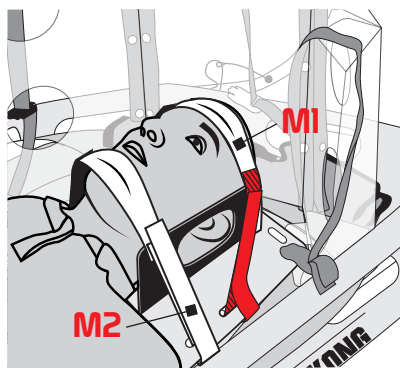
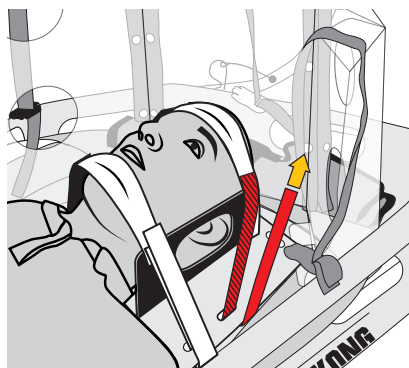
Fig. 23

h) close the watertight sheet with the hinges, attach the buckles of the outer straps and tighten them.

Avoid prolonged contact of the patient's skin with the fabrics of the device

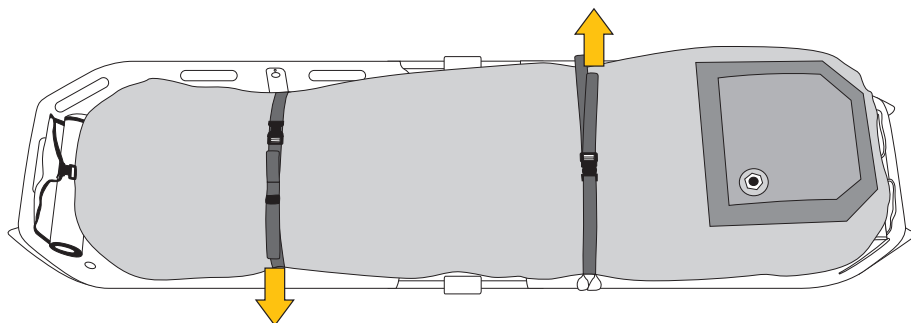
Fig. 24

4.4 TRANSPORTING THE PATIENT

The “911 CANYON” stretcher is suitable for lifting and transporting the patient using the handles (fig. 2), the side handles (fig. 3), suspension (fig. 4-5) and lifting/calculating the patient vertically using the static rope (fig. 6).

4.4.1 Connection with provided handles

a) Connect the handles to the stretcher frame, by using the coupling numbering,

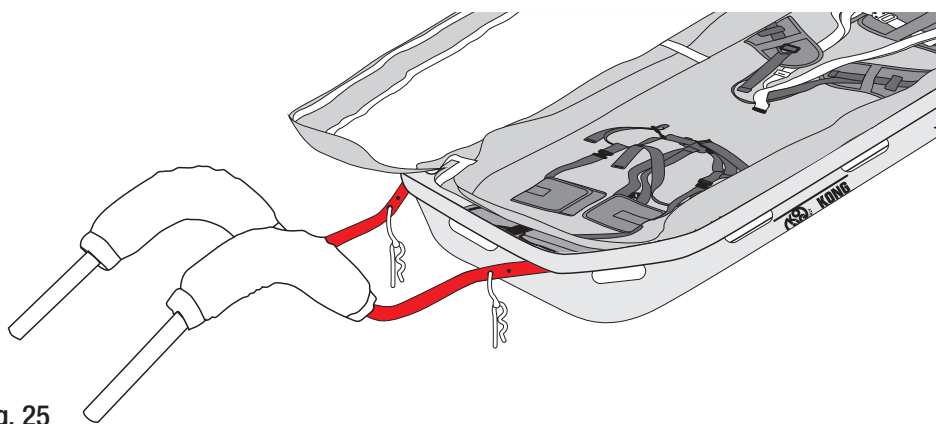
Fig. 25

b) lock the connection by inserting the shaped plug in the hole.

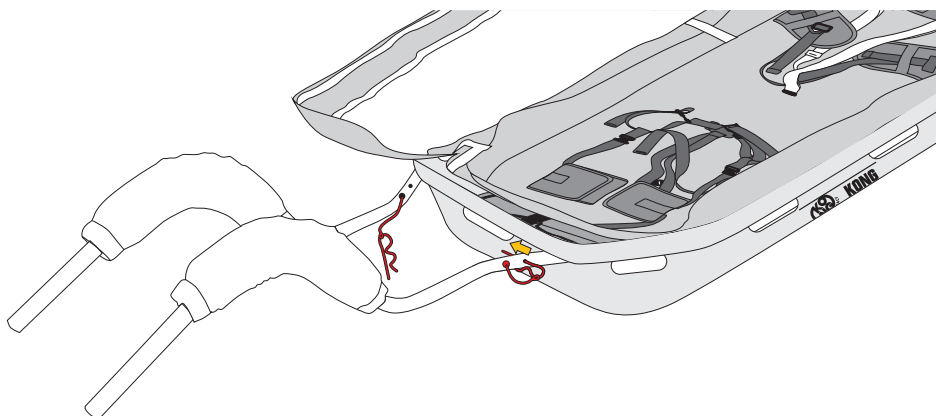
Fig.26

4.4.2 Connection with suspension

- Connect the suspension to the side holes of the stretcher using the four connectors,
- screw the connector nuts in completely.


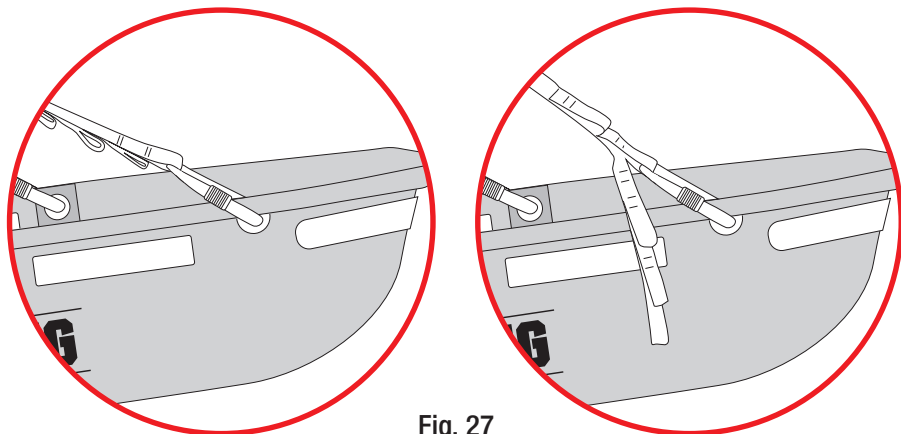
Fig. 27

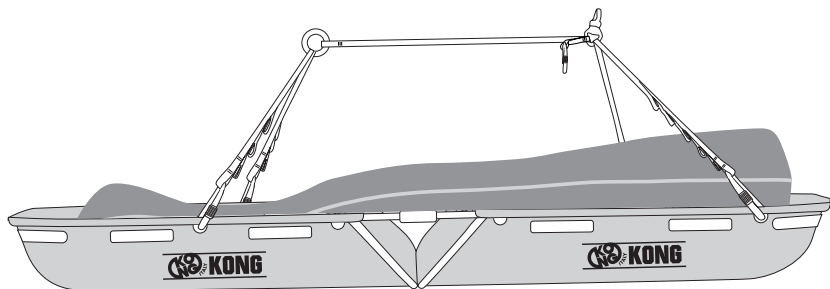The eyelets of the suspension allow you to adjust the position of the stretcher.

4.4.3 Connection with ropes

- Connect the ropes to the attachment points (quick links) at the ends of the stretcher using the connectors,
- screw in the connector nuts completely.

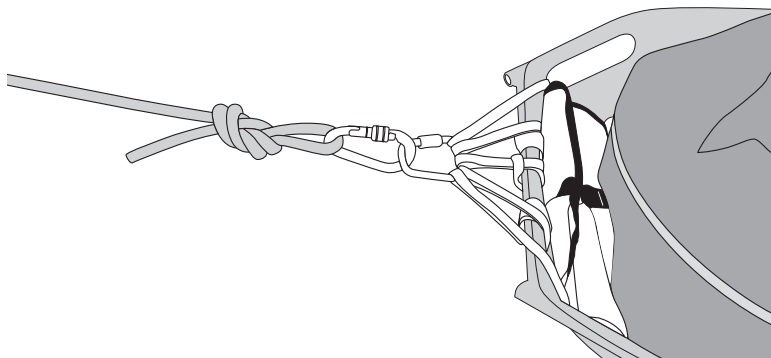
Fig. 28

4.5 TRANSPORTATION OF THE STRETCHER

Attach a shoulder strap to each part of the structure as defined in Figures 29-30 and 31.

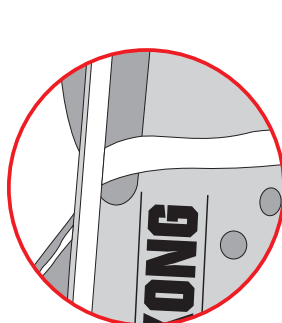
Fig. 29

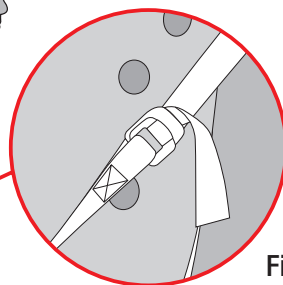
Fig. 30

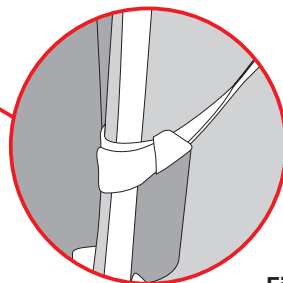
Fig. 31

CHAPTER 5

MAINTENANCE AND REPAIR

5.1 GENERAL INFORMATION

The “**911 CANYON**” stretcher is made of material which is highly resistant to wear and external agents. Despite this, the use conditions require maintenance operations and even repair in particular conditions.

Maintenance and repair operations must be recorded in a special register, an example of which is indicated in chapter 10.

5.2 MAINTENANCE

The maintenance operations that must be carried out by the user are, in order:

- a) cleaning: after every use, wash with warm (max. 40 °C) potable water, possibly adding a neutral detergent (e.g. Marseille soap). Rinse and leave to dry in a shaded place, away from direct sources of heat,
- b) disinfection, when deemed necessary or after prolonged inactivity (more than 21 days), preceded by cleaning: wipe with a clean cloth, soaked in a water solution of sodium hypochlorite (bleach) with a 0,1% concentration (1000 ppm). When blood or other bodily fluids are present, the recommended concentration of sodium hypochlorite is 0,5% (5000ppm).

5.3 REPAIR

Repair shall be strictly carried out by the manufacturer.
The user is allowed to replace the parts mentioned in paragraph 3.4.2 only, with new and genuine parts.

CHAPTER 6

6 STORAGE

After cleaning, any disinfection and drying operations, keep the product and its accessories in a place that is dry (relative humidity 40-90%), cool (temperature 5-40° C) and safe (avoid UV radiation), chemically neutral (absolutely avoid salty and/or acid environments), away from sharp edges, sources of heat, humidity, corrosive substances or other possible jeopardising conditions.

Do not store this device wet!

CHAPTER 7

CHECKS, INSPECTIONS AND REVISIONS

7.1 CHECKS

Before and after every use, check the device and ensure that:

- it is appropriate for the use it was intended for,
- it has not been subjected to mechanical deformations and it neither has cracks nor shows signs of wear,
- the textile parts do not have cuts, burns, chemical products residues, excessive hair, wear, in particular check areas at contact with the metal components (buckles, rings, etc.);
- the markings, including labels, are legible.

7.2 INSPECTIONS

The device must be inspected at least once a year, starting from the date of first use, by expert personnel trained and approved by KONG S.p.A. The date of the first use and the outcome of the inspection must be recorded in the Inspections and Servicing Register, regarding which an example is indicated in chapter 10.

7.3 REVISIONS

The device must be serviced by KONG S.p.A., or by expert personnel trained and approved by KONG S.p.A. if:

- any malfunctions are detected,
- the result of the pre and post use check or of the inspections is negative.

The outcome of the servicing must be recorded in the Inspections and Servicing Register, regarding which an example is indicated in chapter 10.

The revised device is guaranteed for one year from the revision date.

CHAPTER 8

DURATION AND WARRANTY

8.1 DURATION

Carefully read the “Warnings and limitations of use” paragraph in chapter 2. The duration of this device depends on the outcome of the at least one-yearly inspection (paragraph 7.2) and possible servicing. The suspension, even if not used, has a maximum duration of 10 years from the date of manufacture indicated on the label.

Important: any devices that have not passed the pre-use, post-use and periodic inspection must be eliminated and made unusable.

8.2 DISPOSAL

Follow the rules in force in the country of use and the waste disposal procedures of the reference hospital facility for proper disposal.

8.3 WARRANTY

The manufacturer guarantees the compliance of the device with the regulations in force at the time of production. Warranty covering defects is limited to defects regarding raw materials and manufacturing defects. The warranty does not cover normal wear, oxidation, damage caused by non-compliant use and/or in competition, improper maintenance, transportation, preservation or storage. Warranty will be immediately deemed null and void should the device be modified or tampered with. Validity corresponds to the legal validity of the country where the device was sold, as from the date of sale, by the manufacturer. No claims shall be raised against the manufacturer once the aforementioned period has expired. Any repair or replacement request under warranty must be accompanied by a proof of purchase. Should the defect be recognised, the manufacturer undertakes to repair or, replace or refund the device at his discretion. The manufacturer shall not be held liable beyond the invoice price of the device in any case whatsoever.


8.4 LAW OBLIGATIONS

Professional and free time activities are often regulated by special national laws which can impose limits and/or obligations concerning the use of these devices. The user has the obligation to know and apply such laws which could provide for limits different from those indicated in this information.

CHAPTER 9

LABELLING AND SYMBOLS

9.1 MEDICAL DEVICE LABEL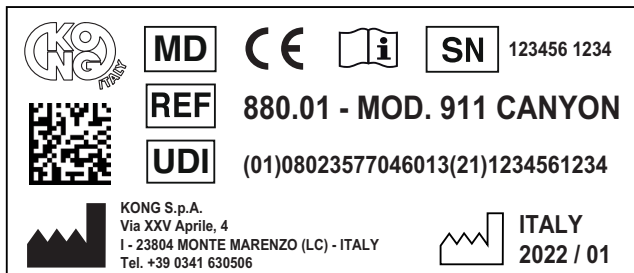


Any changes to the positioning of the symbols does not change the content thereof.

9.2 SYMBOLS

Identification and traceability

	Medical Device
	Product identification code
	Unique Device Identifier
	Manufacturer's identification
	Unique serial number
	Country, year and month of production.

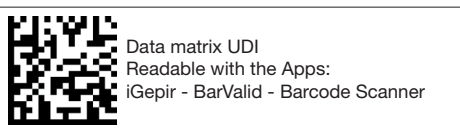Information and safety

	Consult the user manual
	Compliant with Regulation (EU) 2017/745

CHAPTER 10

DOCUMENTS

10.1 INSPECTIONS AND SERVICING REGISTER

STRETCHER - Class I MD				
REF.	880.01	MODEL	911 CANYON	
UDI-DI	08023577046013	SN		
Date of first use				
INSPECTION AND SERVICING				
DATA	I/S	Description	Outcome	Manager

10.2 MAINTENANCE AND REPAIR REGISTER

STRETCHER - Class I MD				
REF.	880.01	MODEL	911 CANYON	
UDI-DI	08023577046013	SN		
Date of first use				
MAINTENANCE AND REPAIR				
DATE	M/R	Description	Outcome	Manager

10.3 EU DECLARATION OF CONFORMITY

The EU Declaration of Conformity is available for download at www.kong.it

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www.kong.it

11.1 APPLIED STANDARDS

- EN 597-1: 2015
Furniture - Assessment of the ignitability of mattresses and upholstered bed bases - Part 1: Ignition source smouldering cigarette
- EN 1865-1: 2010 + A1: 2015
Patient handling equipment used in road ambulances - Part 1: General stretcher systems and patient handling equipment
- EN 62366-1: 2015 + A1: 2020
Medical devices - Application of usability engineering to medical devices
- EN ISO 846:2019
Plastics - Evaluation of the action of microorganisms
- EN ISO 10993-1: 2020
Biological evaluation of medical devices - Part 1: Evaluation and testing within a risk management process
- EN ISO 10993-5: 2009
Biological evaluation of medical devices - Part 5: Tests for in vitro cytotoxicity
- EN ISO 10993-23: 2021
Biological evaluation of medical devices - Part 23: Tests for irritation
- EN ISO 14971: 2019
Medical devices - Application of risk management to medical devices
- EN ISO 15223-1: 2021
Medical devices — Symbols to be used with information to be supplied by the manufacturer — Part 1: General requirements
- EN ISO 20417: 2021
Medical devices - Information to be supplied by the manufacturer.

11.2 STANDARDS USED AS REFERENCE

- EN ISO 13485: 2016
Medical devices - Quality management systems - Requirements for regulatory purposes
- EN 13718-1: 2014 + A1: 2020
Medical vehicles and their equipment - Air ambulances - Part 1: Requirements for medical devices used in air ambulances.

KONG s.p.a.

Via XXV Aprile, 4 - (zona industriale)

I - 23804 MONTE MARENZO (LC) - ITALY

Tel +39 0341630506 - Fax +39 0341641550

www.kong.it